



Authentic[®] Tooth Preparation / Reductions in millimeters

3-12-02


Anterior Veneers

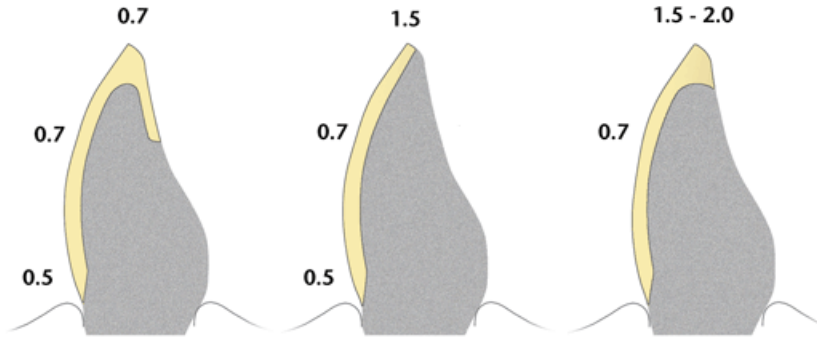

Inlays / Onlays

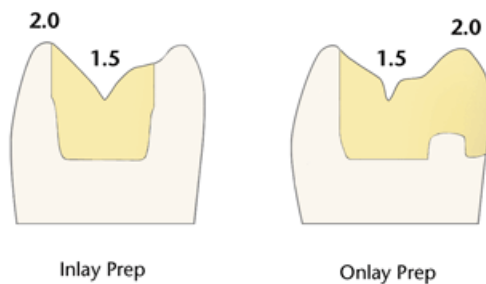

Crowns Prepare 1.5 mm shoulder margins of 90-110° angle.

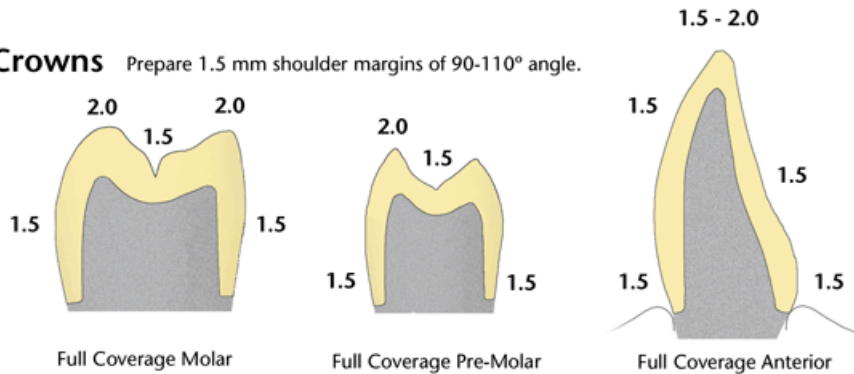

If veneers for anterior teeth are required, the preparation can be very conservative with minimal loss of tooth structure.

First the dentist prepares the enamel layer that shall be covered by the veneer.

For choosing the right color, the patient is advised by the dentist and/or the dental technician, so the laboratory can produce the veneers accordingly.

On delivery to the patient, the restorations are chemically bonded to the tooth structure. Although very little tooth material is eliminated, this method is absolutely strong and durable.

Cementation

All known bonding-luting systems can be used.